

# TECHNICAL DATA SHEET

## FFP2 NR VALVE-FREE MASKS WITH EAR BANDS CERTIFIED AS CATEGORY III PPE



**Manufacturer:**


Xiantao Dingcheng Nonwoven Products Co., Ltd.  
Liukou Industrial Park, Xiantao, Hubei, China.



**Importer/Distributor:**

**Pizzocri S.r.l.**  
Via Fondo Ausa, 28 – 47891 San Marino (RSM)  
[www.pizzocri.com](http://www.pizzocri.com) [info@pizzocri.com](mailto:info@pizzocri.com)

**MAIN PRODUCT DATA**

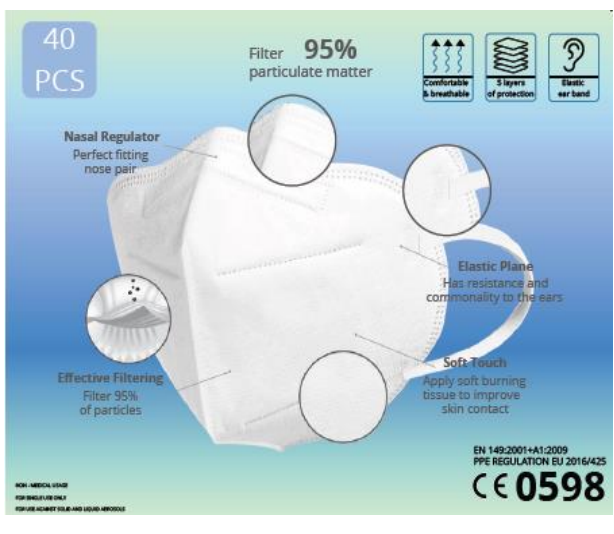

MATERIAL	NON-WOVEN FABRIC - 100% Latex Free
BRAND	DINGCHENG
COLOR	WHITE
MODEL	XTDC-2022-FF02
PRODUCER	Xiantao Dingcheng Nonwoven Products Co., Ltd.
EUROPEAN REPRESENTATIVE	SGS Fimko Ltd
CND	T0399
RDM	2514990
PPE	III CATEGORY
PRODUCTION DATE	08/2024
EXPIRATION DATE	08/2027

**PRODUCT USE AND APPLICATION FIELD**

FFP2 masks are a category of filtering facepieces that offer a high level of respiratory protection. They are designed to filter out fine particles, aerosols, and harmful substances in the air, providing greater protection than surgical masks.

Key protections and uses of FFP2 masks include:

- Protection from fine particles: FFP2 masks are designed to filter at least 95% of airborne particles, including aerosols, fine dust, fumes, and mists.
- High-risk environments: FFP2 masks are often used in environments with a high risk of exposure to hazardous substances such as pathogens, toxic dusts, asbestos, welding fumes, and other harmful fine particles.
- Industrial sectors: They are widely used in the chemical industry, metalworking, construction, pharmaceutical industry, and other work situations where workers' health may be threatened by harmful airborne substances.
- Advanced medical procedures: In some medical situations, such as performing invasive procedures or treating patients with highly contagious diseases, FFP2 masks can be used to protect healthcare workers.
- Individual protection: People with compromised immune systems or medical conditions that make them particularly susceptible to infections may use FFP2 masks for additional protection when exposed to environments with potential pathogens.
- Environmental emergencies: In case of environmental emergencies, such as fires, volcanic eruptions, or other situations where the air is contaminated with toxic substances, FFP2 masks can be used to help people breathe safely.
- USERS: Ideal for all healthcare workers (nurses, doctors, veterinary operators, laboratory technicians, biologists, chemists, physicists, etc.), security personnel, construction/company workers

**SPECIFIC PRODUCT CHARACTERISTICS**

- |  |  |
|--|--|
| <ul style="list-style-type: none"> <li>• Filtering facepieces for protection against infectious agents - FFP2</li> <li>• With mechanical filtration</li> <li>• Equipped with a hypoallergenic patch under the nose clip</li> <li>• Valveless</li> <li>• Made of a latex-free material</li> <li>• Elastics fixed to the mask body without the use of metal staples</li> <li>• Suitable for all face types</li> <li>• Protects the chin, nose, and mouth</li> <li>• Formaldehyde-free</li> <li>• Ergonomic, suitable for all face types</li> <li>• Adjustable nose piece located inside the layers of the nickel-free filter material</li> <li>• Ensures good visibility</li> <li>• Individually packaged to allow for the extraction of a single device at a time, in compliance with hygiene standards.</li> </ul> | <ul style="list-style-type: none"> <li>• Single-use, non-reusable masks, durable for an entire work shift</li> <li>• Biocompatible product</li> <li>• Made of a durable and hypoallergenic material</li> <li>• Design that ensures a perfect fit to the face</li> <li>• Filtration efficiency: filtration rate of non-oily particles ≥ 95%</li> <li>• Comfortable to wear</li> <li>• One size fits all</li> <li>• Protects against harmful solid particles</li> <li>• Suitable for covering the nose, mouth, and chin</li> <li>• Ideal for use in healthcare settings</li> <li>• With double ear loops and a foldable nose piece</li> <li>• Hypoallergenic</li> <li>• Comfortable to wear</li> <li>• High efficiency</li> <li>• Complete with ear loops</li> </ul> |
|--|--|

# CERTIFICATIONS AND TEST REPORT


<p>EC CERTIFICATE</p>	<p>Module B CE certificate: Issued on 02/09/2022 by Notified Body No. 0598 - SGS Fimko Ltd. Certificate n. 0598/PPE/22/3408, conforming to European directive 2016/425.</p> <p>Module D CE certificate: Issued on 14/09/2022 by Notified Body No. 0598 - SGS Fimko Ltd. Certificate n. CN22/00002857, conforming to European directive 2016/425.</p>
<p>DECLARATION OF CONFORMITY ACCORDING TO EUROPEAN REGULATION EU 425/2016.</p>	<p>MANUFACTURER BY</p>
<p>LABORATORY TEST CONFIRMING CONFORMITY TO EUROPEAN STANDARD EN 149:2001+A1:2009</p>	<p>executed by SGS-CSTC Standards Technical Services (Shanghai) Co., Ltd Testing Center, test no. SL52205254133401TX on 29/03/2022.</p>

# PACKAGING

<p>Packaging (1st Level)</p>	<p>1 (one) mask in a sealed plastic bag.</p>
<p>Packaging (2nd Level)</p>	<p>40 (forty) sealed plastic bags (40 masks) in a cardboard box.</p>
<p>Packaging (3rd Level)</p>	<p>50 (fifty) cardboard boxes (2000 masks) in one carton.</p>
<p>Dimensions and weight of the carton</p>	<p>15 kgs / 57*33*67 cm</p>

A carton of 2000 individually packaged masks

<p><b>Mascherine FFP2 NR</b> Senza valvola - Elastici Auricolari Certificata come DPI di III Categoria</p>  <p>Filter 95% particulate matter</p> <p>REF: XTDC-2022-FF02</p> <p>LOT 240820F 20/08/2024 20/08/2022</p> <p>CE 0598</p>	<p>2000 pcs</p> <p>QTY: 40pcs/box 50 boxes/cnt</p> <p>Senza AUSA (Shanghai) Standards Technical Services Co., Ltd. (Lukou Industrial Park, Kunshan, Huabei, China)</p> <p>EN 149:2001+A1:2009 PPE REGULATION EU 2016/425</p> <p>Made in China</p>	<p><b>Mascherine FFP2 NR</b> Senza valvola - Elastici Auricolari Certificata come DPI di III Categoria</p>  <p>Filter 95% particulate matter</p> <p>REF: XTDC-2022-FF02</p> <p>EN 149:2001+A1:2009 PPE REGULATION</p> <p>Made in China</p>	<p>2000 pcs</p> <p>QTY: 40pcs/box 50 boxes/cnt</p> <p>Size: 155mm*110mm G.W.: 15 KGS N.W.: 14 KGS MEAS: 57*33*67cm</p>
--	---	---	--

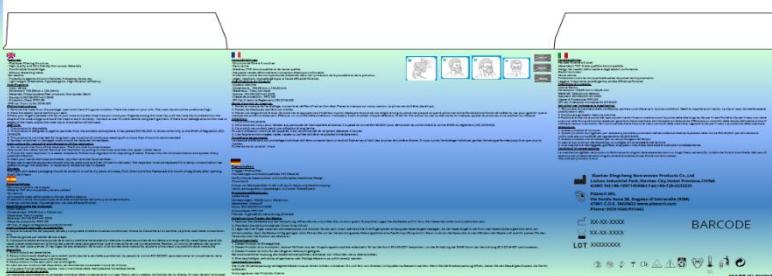

Individual box containing 40 masks

- Each package bears all the information required by current regulations and that necessary to ensure the correct and safe use of the device. The outer packaging is made of a resistant material, suitable for stacking, and indicates the type, quantity, expiration date, and product code. Both the packaging and the individual packages are disposable.
- Each individual box and each individual outer bag bear the data required by Directive 2007/46/EC, Legislative Decree 25 January 2010, no. 37. The intended use is indicated on the packaging.
- The devices are contained in packaging that guarantees their correct storage and easy stacking. The individual packages are packed in sturdy, dustproof boxes. The packaging material is resistant and ensures the correct preservation of the product.
- All individual packages and shipping packages are equipped with indelible labels bearing all the inscriptions required by current regulations.
- The packaging bears, in several languages, all the information required by current regulations and necessary to ensure the correct and safe use of the product.

# EU TYPE-EXAMINATION CERTIFICATE

## Regulation (EU) 2016/425, MODULE B

**0598/PPE/22/3408 Issue 1**  
**0598/PPE/22/3408**

**Product** Filtering half masks  
**Model** XTDC-2022-FF02 Filtering half mask , XTDC-2022 Fish Type Protective mask  
**Trademark** -

**Certificate Holder / Manufacturer** Xiantao Dingcheng Nonwoven Products Co., Ltd.  
 Liukou Industrial Park, Xiantao, Hubei, China

Product complies with the applicable essential health and safety requirements of Regulation (EU) 2016/425 and standard(s) mentioned below

**Standard(s)** EN 149:2001 + A1:2009

**Other Information** Respiratory protective devices. Filtering half masks to protect against particles, with a device classification of FFP2 NR.  
 This certificate shall be used in conjunction with conformity assessment procedure module C2 or D.

**Validity** This certificate is valid until 2027-09-02.

**Date of issue** 2022-09-02


**SGS Fimko Ltd**

**Signature**



Mikko Hirvonen  
 Senior Specialist


SGS Fimko Ltd is a Notified Body (0598) according to the Personal Protective Equipment Regulation (EU)

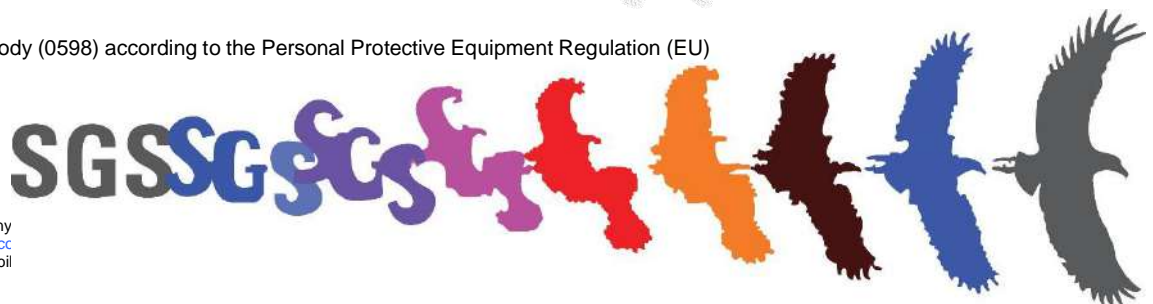

**Additional information** XTDC-2022-FF02 Filtering Half Mask / XTDC-2022 Fish Type Protective mask, consisting of a white five layer (polypropylene) disposable particle filtering half mask, with aluminium / metal silver nose clip, polyester / elastane ear loop.

**Contract No:** SSL-CERT220700105/ GB-202207002405

**Technical File:** Filtering Half Mask XTDC-2022-FF02 / Fish Type Protective mask XTDC-2022, Version 2, Dated: 2022-08-21

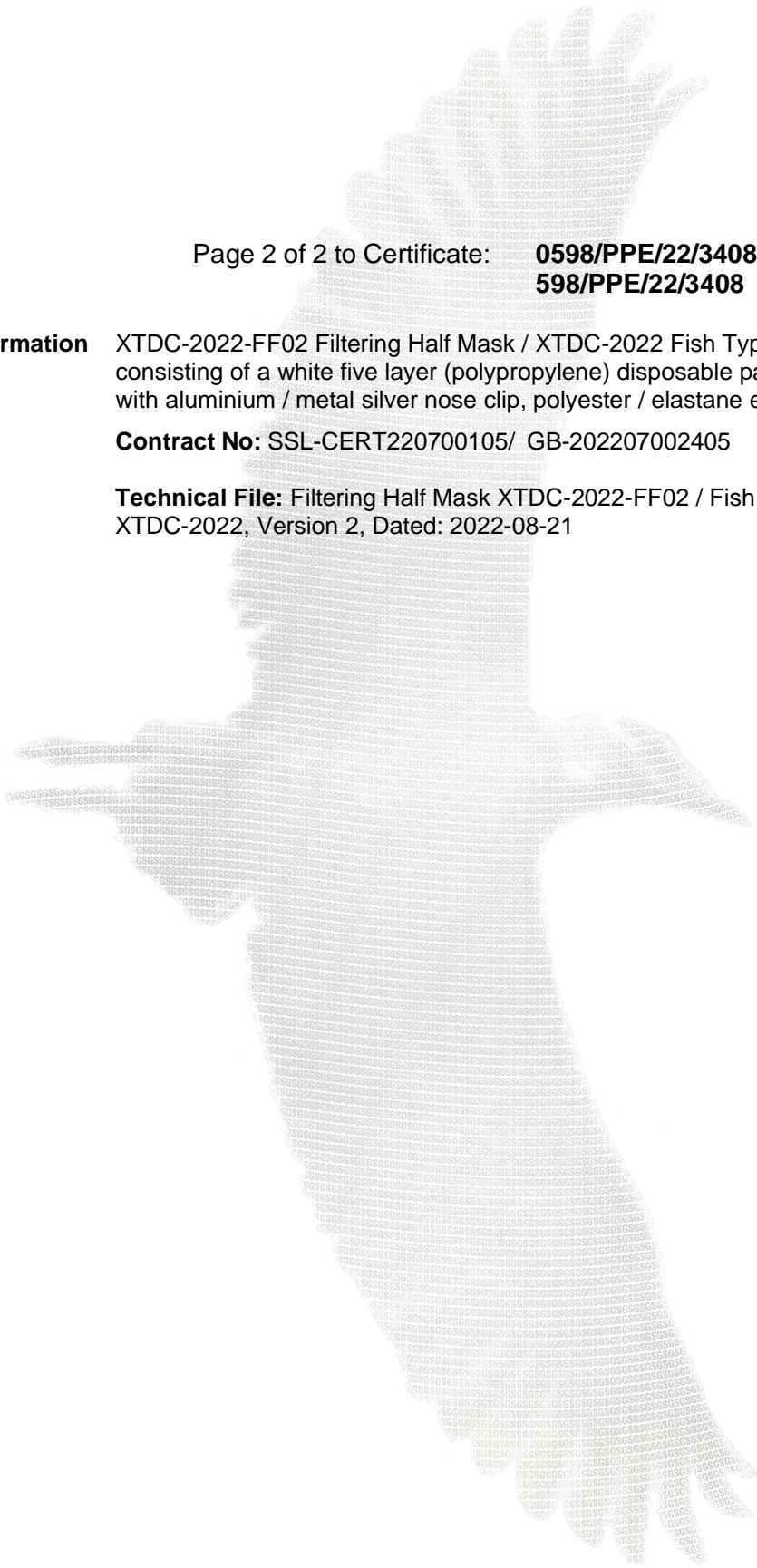

Certificate CN22/00002857

**SGS**

The management system of

# Xiantao Dingcheng Nonwoven Products Co., Ltd.

Liukou Industrial Park, Xiantao City, Hubei Province, China

has been assessed and certified as meeting the requirements of  
**Regulation (EU) 2016/425 Module D**

For the following activities

Manufacture of FFP2 Protective Respirators

(Note: all products marked CE0598 must have a valid EU Type Examination Certificates issued under Module B or a valid EC type examination certificate issued under Article 10 of the PPE Directive 89/686/EEC.)

This certificate is valid from 14 September 2022 until 13 September 2025 and remains valid subject to satisfactory surveillance audits.

Issue 1. Certified since 14 September 2022



Authorised by  
Anna Ruhala

SGS FIMKO OY  
Notified Body 0598 Takomotie 8, FI-00380 Helsinki, Finland  
t +358 9 696 361 - [www.sgs.fi](http://www.sgs.fi)

  
**FINAS**  
Finnish Accreditation Service  
S003 (EN ISO/IEC 17065)

This document is an authentic electronic certificate for Client business purposes use only. Printed version of the electronic certificate are permitted and will be considered as a copy. This document is issued by the Company subject to SGS General Conditions of certification services available on [Terms and Conditions](#) | SGS. Attention is drawn to the limitation of liability, indemnification and jurisdictional clauses contained therein. This document is copyright protected and any unauthorized alteration, forgery or falsification of the content or appearance of this document is unlawful.



XIANTAO DINGCHENG NON-WOVEN PRODUCTS CO., LTD  
LIUKOU INDUSTRIAL PARK 433000 XIANTAO CITY - HUBEI

The following sample(s) was/were submitted and identified on behalf of the client as:

Sample Description : (A) Filtering half mask

Sample Color : (A) White

Composition : (A) Non-woven melt-blown

Style No. : XTDC-2022-FF02

Proposed Care Instruction : -

Test Performed : Selected test(s) as requested by applicant

Sample Receiving Date : Mar 11, 2022

Testing Period : Mar 14, 2022 - Mar 29, 2022

Test Result(s) : Unless otherwise stated the results shown in this test report refer only to the sample(s) tested, for further details, please refer to the following page(s).

**Conclusion:**

Sample No.	Recommendation Level
(A)	FFP2 NR

Signed for and on behalf of  
SGS-CSTC Standards Technical Services (Shanghai) Co., Ltd Testing Center

Sara Guo (Account Executive)

**SGS does not verify authenticity of any Brand/Trademark of products. Buyers must check if the product is genuine with the Brand/Trademark owner directly.**



Unless otherwise agreed in writing, this document is issued by the Company subject to its General Conditions of Service printed overleaf, available on request or accessible at <http://www.sgs.com/en/Terms-and-Conditions.aspx> and, for electronic format documents, subject to Terms and Conditions for Electronic Documents at <http://www.sgs.com/en/Terms-and-Conditions/Terms-e-Documents.aspx>. Attention is drawn to the limitation of liability, indemnification and jurisdiction issues defined therein. Any holder of this document is advised that information contained hereon reflects the Company's findings at the time of its intervention only and within the limits of Client's instructions, if any. The Company's sole responsibility is to its Client and this document does not exonerate parties to a transaction from exercising all their rights and obligations under the transaction documents. This document cannot be reproduced except in full, without prior written approval of the Company. Any unauthorized alteration, forgery or falsification of the content or appearance of this document is unlawful and offenders may be prosecuted to the fullest extent of the law. Unless otherwise stated the results shown in this test report refer only to the sample(s) tested and such sample(s) are retained for 30 days only.

Attention: To check the authenticity of testing/inspection report & certificate, please contact us at telephone: (86-755) 8307 1443, or email: [CN.Doccheck@sgs.com](mailto:CN.Doccheck@sgs.com)

SGS-CSTC Standards Technical Services (Shanghai) Co., Ltd.  
Testing Center

3<sup>rd</sup> Building, No. 889, Yishan Road, Kujui District Shanghai, China 200233  
中国·上海·徐汇区宜山路889号3号楼 邮编: 200233

t (86-21) 61402666 f (86-21) 64958763  
t (86-21) 61402666 f (86-21) 64958763

[www.sgs.com](http://www.sgs.com)  
[sgs.china@sgs.com](mailto:sgs.china@sgs.com)

Test Result

**Personal Protective Equipment - Respiratory Protective Devices- Filtering Half Masks to Protect against Particles- Requirements, Testing, Marking**

EN 149:2001+A1:2009

**Clause 7.4 Packaging**

(EN 149:2001+A1:2009 Clause 8.2)

Test Requirement	Results	Comment
Particle filtering half masks shall be offered for sale packaged in such a way that they are protected against mechanical damage and contamination before use.	Comply	Pass

**Clause 7.5 Material**

(EN 149:2001+A1:2009, Clause 8.2 & 8.3.1 & 8.3.2)

Test Requirement	Results	Comment
Materials used shall be suitable to withstand handling and wear over the period for which the particle filtering half mask is designed to be used.	Comply	Pass
After undergoing the conditioning described in 8.3.1 none of the particle filtering half masks shall have suffered mechanical failure of the facepiece or straps.	Comply	
When conditioned in accordance with 8.3.1 and 8.3.2 the particle filtering half mask shall not collapse.	Comply	
Any material from the filter media released by the air flow through the filter shall not constitute a hazard or nuisance for the wearer.	Comply	

**Clause 7.6 Cleaning and Disinfecting**

(EN 149:2001+A1:2009, Clause 8.4 & 8.5 & 8.11)

Test Requirement	Results	Comment
If the particle filtering half mask is designed to be re-usable, the materials used shall withstand the cleaning and disinfecting agents and procedures to be specified by the manufacturer. With reference to 7.9.2, after cleaning and disinfecting the re-usable particle filtering half mask shall satisfy the penetration requirement of the relevant class.	Not applicable (Not designed to be re-usable)	N.A.

**Clause 7.7 Practical Performance**

(EN 149:2001+A1:2009, Clause 8.4)

Test Requirement	Results	Comment
The particle filtering half mask shall undergo practical performance tests under realistic conditions. These general tests serve the purpose of checking the equipment for imperfections that cannot be determined by the tests described elsewhere in this standard.	No imperfection	Pass



Unless otherwise agreed in writing, this document is issued by the Company subject to its General Conditions of Service printed overleaf, available on request or accessible at <http://www.sgs.com/en/Terms-and-Conditions.aspx> and, for electronic format documents, subject to Terms and Conditions for Electronic Documents at <http://www.sgs.com/en/Terms-and-Conditions/Terms-e-Documents.aspx>. Attention is drawn to the limitation of liability, indemnification and jurisdiction issues defined therein. Any holder of this document is advised that information contained hereon reflects the Company's findings at the time of its intervention only and within the limits of Client's instructions, if any. The Company's sole responsibility is to its Client and this document does not exonerate parties to a transaction from exercising all their rights and obligations under the transaction documents. This document cannot be reproduced except in full, without prior written approval of the Company. Any unauthorized alteration, forgery or falsification of the content or appearance of this document is unlawful and offenders may be prosecuted to the fullest extent of the law. Unless otherwise stated the results shown in this test report refer only to the sample(s) tested and such sample(s) are retained for 30 days only.

Attention: To check the authenticity of testing/inspection report & certificate, please contact us at telephone: (86-755) 8307 1443, or email: [CN.Doccheck@sgs.com](mailto:CN.Doccheck@sgs.com)  
 3<sup>rd</sup> Building No.889, Yishan Road, Xuhui District Shanghai, China 200233 t (86-21) 61402666 f (86-21) 64958763 www.sgs.com.cn  
 中国·上海·徐汇区宜山路889号3号楼 邮编: 200233 t (86-21) 61402666 f (86-21) 64958763 e [sgs.china@sgs.com](mailto:sgs.china@sgs.com)

**Clause 7.8 Finish of Parts**

(EN 149:2001+A1:2009, Clause 8.2)

Test Requirement	Results	Comment
Parts of the device likely to come into contact with the wearer shall have no sharp edges or burrs.	No sharp edges or burrs	Pass

**Clause 7.9.1 Total Inward Leakage**

(EN 149:2001+A1:2009, Clause 8.5)

Test Requirement	Results	Comment
<p>The total inward leakage consists of three components: face seal leakage, exhalation value leakage(if exhalation value fitted) and filter penetration. For particle filtering half masks fitted in accordance with the manufacturer's information, at least 46 out of the 50 individual exercise results (i.e. 10 subjects x 5 exercises) for total inward leakage shall be not greater than: 25% for FFP1, 11% for FFP2, 5% for FFP3</p> <p>and, in addition, at least 8 out of the 10 individual wearer arithmetic means for the total inward leakage shall be not greater than: 22% for FFP1, 8% for FFP2, 2% for FFP3</p>	Detail refer to Appendix 1	Meet FFP1 Meet FFP2

**Appendix 1: Summarization of Test Data**

**Inward Leakage Test Data**

Subject	Sample No.	Condition	Walk(%)	Head Side/side(%)	Head up/down(%)	Talk(%)	Walk(%)	Mean(%)
Zhou	1	A.R.	4.23	4.43	4.17	4.56	4.20	4.32
Luo	2	A.R.	4.52	5.19	5.00	5.54	5.40	5.13
Lu	3	A.R.	4.36	4.74	4.28	4.90	4.19	4.49
Wang	4	A.R.	3.38	3.10	3.02	3.44	3.38	3.26
Bao	5	A.R.	5.39	4.25	5.30	5.99	5.70	5.33
Ding	6	T.C.	3.01	3.43	3.58	4.28	3.22	3.50
Li	7	T.C.	5.63	5.10	5.80	6.05	5.03	5.52
Chen	8	T.C.	3.44	3.33	3.94	4.05	3.55	3.66
Song	9	T.C.	4.55	4.46	4.75	5.11	4.74	4.72
Ye	10	T.C.	5.44	6.01	5.17	6.22	5.19	5.61

**Facial Dimension**

Subject	Face length	Face Width	Face Depth	Mouth Width
Chen	125	150	120	58
Lu	115	132	107	48
Zhou	115	135	106	52
Li	125	130	107	46
Luo	125	136	100	43
Zheng	128	140	112	55
Wang	120	147	103	48
Song	120	140	100	50
Bao	130	134	104	50
Ding	134	150	110	52

Unless otherwise agreed in writing, this document is issued by the Company subject to its General Conditions of Service printed overleaf, available on request or accessible at <http://www.sgs.com/en/Terms-and-Conditions.aspx> and, for electronic format documents, subject to Terms and Conditions for Electronic Documents at <http://www.sgs.com/en/Terms-and-Conditions/Terms-e-Documents.aspx>. Attention is drawn to the limitation of liability, indemnification and jurisdiction issues defined therein. Any holder of this document is advised that information contained hereon reflects the Company's findings at the time of its intervention only and within the limits of Client's instructions, if any. The Company's sole responsibility is to its Client and this document does not exonerate parties to a transaction from exercising all their rights and obligations under the transaction documents. This document cannot be reproduced except in full, without prior written approval of the Company. Any unauthorized alteration, forgery or falsification of the content or appearance of this document is unlawful and offenders may be prosecuted to the fullest extent of the law. Unless otherwise stated the results shown in this test report refer only to the sample(s) tested and such sample(s) are retained for 30 days only.

Attention: To check the authenticity of testing/inspection report & certificate, please contact us at telephone: (86-755) 8307 1443, or email: [CN.Doccheck@sgs.com](mailto:CN.Doccheck@sgs.com)

3<sup>rd</sup> Building, No. 889, Yishan Road, Xuhui District Shanghai, China 200233 t (86-21) 61402666 f (86-21) 64958763 www.sgs.com.cn  
 中国·上海·徐汇区宜山路889号3号楼 邮编: 200233 t (86-21) 61402666 f (86-21) 64958763 e [sgs.china@sgs.com](mailto:sgs.china@sgs.com)



Liu	120	135	117	50
Ye	126	137	105	52

Note: A panel of ten clean-shaven persons (without beards or sideburns) were selected covering the spectrum of facial characteristics of typical users of the filtering half masks submitted by applicant (excluding significant abnormalities). It is to be expected that exceptionally some persons cannot be satisfactorily fitted with a particle filtering half mask. Such exceptional subjects shall not be used for testing particle filtering half masks.

**Clause 7.9.2 Penetration of Filter Material**  
(EN 149:2001+A1:2009, Clause 8.11 & EN 13274-7:2019)

Test Requirement			Results	Comment
The penetration of the filter of the particle filtering half mask shall meet the requirements of the following table.			Detail refer to Appendix 2	Meet FFP1 Meet FFP2 Meet FFP3
Classification	Maximum penetration of test aerosol			
	Sodium chloride test 95 l/min %	Paraffin oil test 95 l/min %		
	max.	max.		
FFP1	20	20		
FFP2	6	6		
FFP3	1	1		

**Appendix 2: Summarization of Test Data**

Penetration of filter material

Aerosol	Condition	Sample No.	Penetration (%)
Sodium chloride test	Penetration test (As received)	1	0.121
		2	0.110
		3	0.137
	Penetration test (Simulated wearing treatment)	4	0.146
		5	0.125
		6	0.138
	Exposure test (Mechanical strength + Temperature conditioned)	7	0.235
		8	0.279
		9	0.262
Paraffin oil test	Penetration test (As received)	10	0.201
		11	0.179
		12	0.225
	Penetration test (Simulated wearing treatment)	13	0.214
		14	0.222
		15	0.208
	Exposure test (Mechanical strength + Temperature conditioned)	16	0.393
		17	0.384
		18	0.405
Flow conditioning: Single filter: 95.0 L/min			



Unless otherwise agreed in writing, this document is issued by the Company subject to its General Conditions of Service printed overleaf, available on request or accessible at <http://www.sgs.com/en/Terms-and-Conditions.aspx> and, for electronic format documents, subject to Terms and Conditions for Electronic Documents at <http://www.sgs.com/en/Terms-and-Conditions/Terms-e-Documents.aspx>. Attention is drawn to the limitation of liability, indemnification and jurisdiction issues defined therein. Any holder of this document is advised that information contained hereon reflects the Company's findings at the time of its intervention only and within the limits of Client's instructions, if any. The Company's sole responsibility is to its Client and this document does not exonerate parties to a transaction from exercising all their rights and obligations under the transaction documents. This document cannot be reproduced except in full, without prior written approval of the Company. Any unauthorized alteration, forgery or falsification of the content or appearance of this document is unlawful and offenders may be prosecuted to the fullest extent of the law. Unless otherwise stated the results shown in this test report refer only to the sample(s) tested and such sample(s) are retained for 30 days only.

Attention: To check the authenticity of testing/inspection report & certificate, please contact us at telephone: (86-755) 8307 1443, or email: [CN.Doccheck@sgs.com](mailto:CN.Doccheck@sgs.com)  
 3<sup>rd</sup> Building, No. 889, Yishan Road, Xuhui District Shanghai, China 200233 t (86-21) 61402666 f (86-21) 64958763 www.sgs.com.cn  
 中国·上海·徐汇区宜山路889号3号楼 邮编: 200233 t (86-21) 61402666 f (86-21) 64958763 e [sgs.china@sgs.com](mailto:sgs.china@sgs.com)

**Clause 7.10 Compatibility with Skin**

(EN 149:2001+A1:2009, Clause 8.4 & 8.5)

Test Requirement	Results	Comment
Materials that may come into contact with the wearer's skin shall not be known to be likely to cause irritation or any other adverse effect to health.	No irritation or any other adverse effect to health	Pass

**Clause 7.11 Flammability**

(EN 149:2001+A1:2009, Clause 8.6)

A	No. 1	No. 2	Requirement
<u>As Received</u> After Flame Time(s)	0	0	Max.5s
<u>Temperature Conditioned</u> After Flame Time(s)	0	0	Max.5s

Remark: Test shall be performed on 2 individual particle filtering half mask for each conditioning (as received & temperature conditioned)

**Clause 7.12 Carbon Dioxide Content of The Inhalation Air**

(EN 149:2001+A1:2009, Clause 8.7)

Test Requirement	Results	Comment
The carbon dioxide content of the inhalation air (dead space) shall not exceed an average of 1,0 % (by volume)	Detail refer to Appendix 4	Pass

**Appendix 4: Summarization of Test Data**

Carbon Dioxide Content of The Inhalation Air

Condition	Sample No.	Result
As received	1	0.5621
	2	0.5634
	3	0.5633
		Mean value:0.56

**Clause 7.13 Head Harness**

(EN 149:2001+A1:2009, Clause 8.4 & 8.5)

Test Requirement	Results	Comment
The head harness shall be designed so that the particle filtering half mask can be donned and removed easily.	Comply	Pass
The head harness shall be adjustable or self-adjusting and shall be sufficiently robust to hold the particle filtering half mask firmly in position and be capable of maintaining total inward leakage requirements for the device.	Comply	



Unless otherwise agreed in writing, this document is issued by the Company subject to its General Conditions of Service printed overleaf, available on request or accessible at <http://www.sgs.com/en/Terms-and-Conditions.aspx> and, for electronic format documents, subject to Terms and Conditions for Electronic Documents at <http://www.sgs.com/en/Terms-and-Conditions/Terms-e-Documents.aspx>. Attention is drawn to the limitation of liability, indemnification and jurisdiction issues defined therein. Any holder of this document is advised that information contained hereon reflects the Company's findings at the time of its intervention only and within the limits of Client's instructions, if any. The Company's sole responsibility is to its Client and this document does not exonerate parties to a transaction from exercising all their rights and obligations under the transaction documents. This document cannot be reproduced except in full, without prior written approval of the Company. Any unauthorized alteration, forgery or falsification of the content or appearance of this document is unlawful and offenders may be prosecuted to the fullest extent of the law. Unless otherwise stated the results shown in this test report refer only to the sample(s) tested and such sample(s) are retained for 30 days only.

Attention: To check the authenticity of testing/inspection report & certificate, please contact us at telephone: (86-755) 8307 1443, or email: [CN.Doccheck@sgs.com](mailto:CN.Doccheck@sgs.com)

3<sup>rd</sup> Building, No. 889, Yishan Road, Xuhui District Shanghai, China 200233 t (86-21) 61402666 f (86-21) 64958763 www.sgs.com.cn  
 中国·上海·徐汇区宜山路889号3号楼 邮编: 200233 t (86-21) 61402666 f (86-21) 64958763 e [sgs.china@sgs.com](mailto:sgs.china@sgs.com)

**Clause 7.14 Field of Vision**

(EN 149:2001+A1:2009, Clause 8.4)

Test Requirement	Results	Comment
The field of vision is acceptable if determined so in practical performance tests.	Comply	Pass

**Clause 7.15 Exhalation Valve(s)**

(EN 149:2001+A1:2009, Clause 8.2 & 8.9.1 & 8.3.4 & 8.8)

Test Requirement	Results	Comment
(a) A particle filtering half mask may have one or more exhalation valve(s), which shall function correctly in all orientations.	Not applicable due to No exhalation valve	N.A.
(b) If an exhalation valve is provided it shall be protected against or be resistant to dirt and mechanical damage and may be shrouded or may include any other device that may be necessary for the particle filtering half mask to comply with 7.9.	Not applicable due to No exhalation valve	
(c) Exhalation valve(s), if fitted, shall continue to operate correctly after a continuous exhalation flow of 300 l/min over a period of 30 s.	Not applicable due to No exhalation valve	
(d) When the exhalation valve housing is attached to the faceblank, it shall withstand axially a tensile force of 10N applied for 10 s.	Not applicable due to No exhalation valve	

**Clause 7.16 Breathing Resistance**

(EN 149:2001+A1:2009, Clause 8.9)

Test Requirement	Results	Comment																						
The breathing resistance of the filter of the particle filtering half mask shall meet the requirements of the following table.	Detail refer to Appendix 5	Meet FFP1 Meet FFP2 Meet FFP3																						
<table border="1"> <thead> <tr> <th rowspan="3">Classification</th> <th colspan="3">Maximum permitted resistance (mbar)</th> </tr> <tr> <th colspan="2">Inhalation</th> <th>Exhalation</th> </tr> <tr> <th>30 l/min</th> <th>95 l/min</th> <th>160 l/min</th> </tr> </thead> <tbody> <tr> <td>FFP1</td> <td>0.6</td> <td>2.1</td> <td>3.0</td> </tr> <tr> <td>FFP2</td> <td>0.7</td> <td>2.4</td> <td>3.0</td> </tr> <tr> <td>FFP3</td> <td>1.0</td> <td>3.0</td> <td>3.0</td> </tr> </tbody> </table>			Classification	Maximum permitted resistance (mbar)			Inhalation		Exhalation	30 l/min	95 l/min	160 l/min	FFP1	0.6	2.1	3.0	FFP2	0.7	2.4	3.0	FFP3	1.0	3.0	3.0
Classification				Maximum permitted resistance (mbar)																				
				Inhalation		Exhalation																		
			30 l/min	95 l/min	160 l/min																			
FFP1	0.6	2.1	3.0																					
FFP2	0.7	2.4	3.0																					
FFP3	1.0	3.0	3.0																					

**Appendix 5: Summarization of Test Data**



Unless otherwise agreed in writing, this document is issued by the Company subject to its General Conditions of Service printed overleaf, available on request or accessible at <http://www.sgs.com/en/Terms-and-Conditions.aspx> and, for electronic format documents, subject to Terms and Conditions for Electronic Documents at <http://www.sgs.com/en/Terms-and-Conditions/Terms-e-Documents.aspx>. Attention is drawn to the limitation of liability, indemnification and jurisdiction issues defined therein. Any holder of this document is advised that information contained hereon reflects the Company's findings at the time of its intervention only and within the limits of Client's instructions, if any. The Company's sole responsibility is to its Client and this document does not exonerate parties to a transaction from exercising all their rights and obligations under the transaction documents. This document cannot be reproduced except in full, without prior written approval of the Company. Any unauthorized alteration, forgery or falsification of the content or appearance of this document is unlawful and offenders may be prosecuted to the fullest extent of the law. Unless otherwise stated the results shown in this test report refer only to the sample(s) tested and such sample(s) are retained for 30 days only.

Attention: To check the authenticity of testing/inspection report & certificate, please contact us at telephone: (86-755) 8307 1443, or email: [CN.Doccheck@sgs.com](mailto:CN.Doccheck@sgs.com)

SGS-CSI (Shanghai) Technical Services (Shanghai) Co., Ltd. 3<sup>rd</sup> Building, No. 889, Yishan Road, Xuhui District Shanghai, China 200233 t (86-21) 61402666 f (86-21) 64958763 www.sgs.com.cn  
 Testing Center: 中国·上海·徐汇区宜山路889号3号楼 邮编: 200233 t (86-21) 61402666 f (86-21) 64958763 e [sgs.china@sgs.com](mailto:sgs.china@sgs.com)

Breathing resistance (mbar)

As received	Flow rate		1					2					3				
			A	B	C	D	E	A	B	C	D	E	A	B	C	D	E
	Inhalation	30 l/min	0.4	0.3	0.4	0.4	0.4	0.4	0.3	0.4	0.3	0.3	0.4	0.4	0.4	0.4	0.4
	95 l/min	1.4	1.4	1.4	1.4	1.3	1.4	1.3	1.3	1.4	1.4	1.3	1.4	1.3	1.4	1.4	
Exhalation	160 l/min	2.5	2.5	2.4	2.4	2.5	2.5	2.5	2.4	2.4	2.5	2.5	2.4	2.5	2.5	2.5	
Simulated wearing treatment	Flow rate		4					5					6				
			A	B	C	D	E	A	B	C	D	E	A	B	C	D	E
	Inhalation	30 l/min	0.4	0.4	0.3	0.3	0.3	0.4	0.3	0.4	0.4	0.4	0.3	0.3	0.4	0.4	0.3
		95 l/min	1.4	1.3	1.4	1.4	1.4	1.3	1.4	1.3	1.3	1.3	1.4	1.3	1.4	1.4	1.4
	Exhalation	160 l/min	2.4	2.5	2.5	2.5	2.5	2.4	2.4	2.5	2.4	2.4	2.4	2.4	2.5	2.4	2.5
Temperature conditioned	Flow rate		7					8					9				
			A	B	C	D	E	A	B	C	D	E	A	B	C	D	E
	Inhalation	30 l/min	0.3	0.3	0.4	0.3	0.3	0.4	0.4	0.3	0.3	0.3	0.3	0.4	0.3	0.3	0.3
		95 l/min	1.4	1.3	1.3	1.3	1.3	1.4	1.4	1.3	1.4	1.3	1.3	1.4	1.3	1.3	1.3
	Exhalation	160 l/min	2.4	2.4	2.4	2.4	2.5	2.4	2.4	2.4	2.4	2.5	2.5	2.5	2.4	2.4	2.4

A: facing directly ahead; B: facing vertically upwards; C: facing vertically downwards; D: lying on the left side; E: lying on the right side

**Clause 7.17 Clogging**

(EN 149:2001+A1:2009, Clause 8.9 & 8.10)

Test Requirement	Results	Comment																			
<p><u>Clause 7.17.2 Breathing resistance</u>  <u>Valved particle filtering half masks:</u>                      After clogging the inhalation resistances shall not exceed:                      FFP1: 4 mbar, FFP2: 5 mbar, FFP3: 7 mbar at 95L/min continuous flow                      The exhalation resistance shall not exceed 3 mbar at 160 L/min continuous flow.</p> <p><u>Valveless particle filtering half masks:</u>                      After clogging the inhalation and exhalation resistances shall not exceed:                      FFP1: 3 mbar, FFP2: 4 mbar, FFP3: 5 mbar at 95L/min continuous flow</p>	Optional for single shift device only	N.A.																			
<p><u>Clause 7.17.3 Penetration of filter material</u>                      All types (valved and valveless) of particle filtering half masks claimed to meet the clogging requirement shall also meet the requirements.</p> <table border="1"> <thead> <tr> <th rowspan="3">Classification</th> <th colspan="2">Maximum penetration of test aerosol</th> </tr> <tr> <th>Sodium chloride test 95 l/min</th> <th>Paraffin oil test 95 l/min</th> </tr> <tr> <th>%</th> <th>%</th> </tr> </thead> <tbody> <tr> <td></td> <td>max.</td> <td>max.</td> </tr> <tr> <td>FFP1</td> <td>20</td> <td>20</td> </tr> <tr> <td>FFP2</td> <td>6</td> <td>6</td> </tr> <tr> <td>FFP3</td> <td>1</td> <td>1</td> </tr> </tbody> </table>	Classification	Maximum penetration of test aerosol		Sodium chloride test 95 l/min	Paraffin oil test 95 l/min	%	%		max.	max.	FFP1	20	20	FFP2	6	6	FFP3	1	1	Optional for single shift device only	N.A.
Classification		Maximum penetration of test aerosol																			
		Sodium chloride test 95 l/min	Paraffin oil test 95 l/min																		
	%	%																			
	max.	max.																			
FFP1	20	20																			
FFP2	6	6																			
FFP3	1	1																			




Unless otherwise agreed in writing, this document is issued by the Company subject to its General Conditions of Service printed overleaf, available on request or accessible at <http://www.sgs.com/en/Terms-and-Conditions.aspx> and, for electronic format documents, subject to Terms and Conditions for Electronic Documents at <http://www.sgs.com/en/Terms-and-Conditions/Terms-e-Documents.aspx>. Attention is drawn to the limitation of liability, indemnification and jurisdiction issues defined therein. Any holder of this document is advised that information contained hereon reflects the Company's findings at the time of its intervention only and within the limits of Client's instructions, if any. The Company's sole responsibility is to its Client and this document does not exonerate parties to a transaction from exercising all their rights and obligations under the transaction documents. This document cannot be reproduced except in full, without prior written approval of the Company. Any unauthorized alteration, forgery or falsification of the content or appearance of this document is unlawful and offenders may be prosecuted to the fullest extent of the law. Unless otherwise stated the results shown in this test report refer only to the sample(s) tested and such sample(s) are retained for 30 days only.

Attention: To check the authenticity of testing/inspection report & certificate, please contact us at telephone: (86-755) 8307 1443, or email: [CN.Doccheck@sgs.com](mailto:CN.Doccheck@sgs.com)

SGS-CS (Shanghai) Technical Services (Shanghai) Co., Ltd. 3<sup>rd</sup> Building, No. 889, Yishan Road, Xuhui District Shanghai, China 200233 t (86-21) 61402666 f (86-21) 64958763 www.sgs.com.cn  
 Testing Center: 中国·上海·徐汇区宜山路889号3号楼 邮编: 200233 t (86-21) 61402666 f (86-21) 64958763 e [sgs.china@sgs.com](mailto:sgs.china@sgs.com)



### Sample Photo

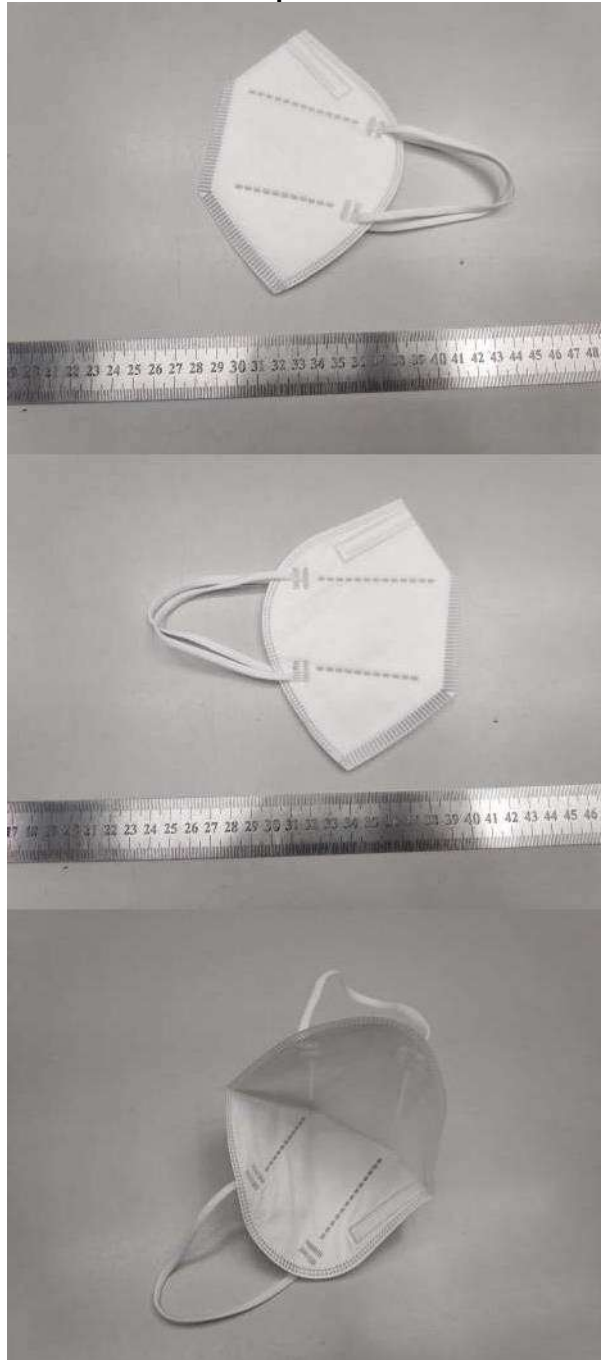


Unless otherwise agreed in writing, this document is issued by the Company subject to its General Conditions of Service printed overleaf, available on request or accessible at <http://www.sgs.com/en/Terms-and-Conditions.aspx> and, for electronic format documents, subject to Terms and Conditions for Electronic Documents at <http://www.sgs.com/en/Terms-and-Conditions/Terms-e-Documents.aspx>. Attention is drawn to the limitation of liability, indemnification and jurisdiction issues defined therein. Any holder of this document is advised that information contained hereon reflects the Company's findings at the time of its intervention only and within the limits of Client's instructions, if any. The Company's sole responsibility is to its Client and this document does not exonerate parties to a transaction from exercising all their rights and obligations under the transaction documents. This document cannot be reproduced except in full, without prior written approval of the Company. Any unauthorized alteration, forgery or falsification of the content or appearance of this document is unlawful and offenders may be prosecuted to the fullest extent of the law. Unless otherwise stated the results shown in this test report refer only to the sample(s) tested and such sample(s) are retained for 30 days only.

Attention: To check the authenticity of testing/inspection report & certificate, please contact us at telephone: (86-755) 8307 1443, or email: [CN.Doccheck@sgs.com](mailto:CN.Doccheck@sgs.com)

3<sup>rd</sup> Building, No. 889, Yishan Road, Xuhui District Shanghai, China 200233 t (86-21) 61402666 f (86-21) 64958763 www.sgs.com.cn  
中国·上海·徐汇区宜山路889号3号楼 邮编: 200233 t (86-21) 61402666 f (86-21) 64958763 e [sgs.china@sgs.com](mailto:sgs.china@sgs.com)



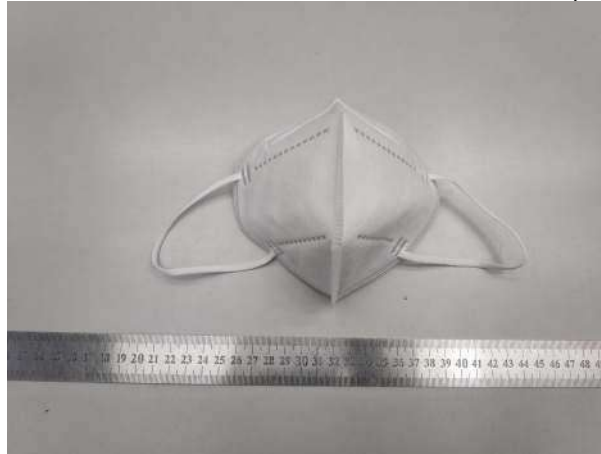

**Product information is provided by applicant without verification or authentication of the brand.**

\*\*\*End of Report\*\*\*



Unless otherwise agreed in writing, this document is issued by the Company subject to its General Conditions of Service printed overleaf, available on request or accessible at <http://www.sgs.com/en/Terms-and-Conditions.aspx> and, for electronic format documents, subject to Terms and Conditions for Electronic Documents at <http://www.sgs.com/en/Terms-and-Conditions/Terms-e-Documents.aspx>. Attention is drawn to the limitation of liability, indemnification and jurisdiction issues defined therein. Any holder of this document is advised that information contained hereon reflects the Company's findings at the time of its intervention only and within the limits of Client's instructions, if any. The Company's sole responsibility is to its Client and this document does not exonerate parties to a transaction from exercising all their rights and obligations under the transaction documents. This document cannot be reproduced except in full, without prior written approval of the Company. Any unauthorized alteration, forgery or falsification of the content or appearance of this document is unlawful and offenders may be prosecuted to the fullest extent of the law. Unless otherwise stated the results shown in this test report refer only to the sample(s) tested and such sample(s) are retained for 30 days only.

Attention: To check the authenticity of testing/inspection report & certificate, please contact us at telephone: (86-755) 8307 1443, or email: [CN.Doccheck@sgs.com](mailto:CN.Doccheck@sgs.com)

	<b>INSPECTION REPORT</b> 检验报告 <i>THIS DOCUMENT IS NON-NEGOTIABLE</i> 此份报告不作为对单据	<b>Inspection No.</b> 检验编号:	3264758	
		<b>Trade Assurance No.</b> 信保订单号:	197167851501022392	
		<b>Report Date</b> 报告日期:	Sep 14, 2024	Page 1 of 17

<b>A. GENERAL 常规信息</b>				
To 至:	Alibaba.com	Vendor/supplier Name: Xiantao Dingcheng Nonwoven Products Co., Ltd. 供应商名称:		
Supplier contact/Tel: 供应商联系人/电话:	Jeff /18971459684			
Inspection Date: 检验日期:	Sep 13, 2024	Product Description: Disposable FFP2 mask 产品描述:		
Inspector Name(s): 检验员姓名:	Tang Jianwu			
Supervisor Name(s): 监督审核员:	Wang Tingqiang			
Inspection Location: 检验地址:	Xiantao City, Hubei Province, China			

<b>B. INSPECTION RESULT SUMMARY 检验结果汇总</b>					
			<b>PASS 合格</b>	<b>FAIL 不合格</b>	<b>REMARK 备注</b>
1.	Conformity Check 一致性检查	Quantity 数量	X		
2.		Shipping Marks 唛头	X		
3.		Export carton packing 出口外包装	X		
4.		Inner carton packing 内包装	X		
5.		Product type (including style, size, marking, label, printings etc.) 产品类型(包括规格、尺寸、标记、标签、印刷等)	X		
6.		Workmanship / Appearance / Function 工艺 / 外观 / 功能	X		
7.		On-site Tests 现场测试	X		

<b>C. REMARKS 备注</b>	
1	The inventory check of the quantity of goods on site is based on the packing list data signed by the factory.

<b>D. OVERALL RESULT 最终检验结果</b> <input checked="" type="checkbox"/> PASS 合格 <input type="checkbox"/> FAIL 不合格
--

If you have any questions about the inspection result, please do not hesitate to contact us.  
 Email address: [yh@ccicbj.com](mailto:yh@ccicbj.com)

The above reflects our findings at the time and place of inspection based on comparison of your sample with random samples selected in accordance with a previously agreed random statistical standard. This report/certificate does not discharge or release sellers/suppliers from their commercial, legal or contractual obligations with regard to quality/quantity of delivery nor does it prejudice the buyer's right of claim toward sellers/suppliers for compensation for any apparent and/hidden defects not detected during our random inspection or occurring thereafter. 上述检查发现反映了在检验时间和地点的基于按照选定的随机抽样样品进行比较和预定的随机统计标准的结果。本报告/证书不能解除或减轻卖方和供应商的商业、法律或合同义务；不影响买方对交付产品的质量/数量过程中存在的以任何明显的和隐藏缺陷向卖方/供应商的索赔。无论是在我们随机检查或此后交付的过程中。

<b>INSPECTION REPORT</b> 检验报告 <i>THIS DOCUMENT IS NON-NEGOTIABLE</i> 此份报告不作为议单据	<b>Inspection No.</b> 检验编号:	3264758	
	<b>Trade Assurance No.</b> 信保订单号:	197167851501022392	
	<b>Report Date</b> 报告日期:	Sep 14, 2024	Page 2 of 17

**E. WORKMANSHIP(Basic Function & Appearance) 工艺(基本功能和外观)**

Inspection Standard 检验标准: ANSI/ASQ Z1.4 (formerly known as MIL-STD-105E)					
Inspection Level 检验级别 :	I I	AQL - Critical Defectives 致命缺陷:	Not allowed 不允许		
Total number of cartons opened 总开箱数量:	23	AQL - Major Defectives 主要缺陷:	2.5		
		AQL - Minor Defectives 次要缺陷::	4.0		
Item No. 产品型号:	Defective Description 缺陷描述		Critical 致命	Major 主要	Minor 次要
XTDC-2022-FF02	Insecure loop		0	1	0
	Dirt mark at loop		0	0	3
<b>Total Found总发现个数:</b>			<b>0</b>	<b>1</b>	<b>3</b>
<b>Allowed允许个数:</b>			<b>0</b>	<b>14</b>	<b>21</b>
<b>Result (PASS/FAIL) 结果(合格/不合格):</b>			<b>Pass</b>	<b>Pass</b>	<b>Pass</b>







**F. QUANTITY数量**

Item/Style/ Product No. 型号/规格/产品号	Order Quantity 订单数量	Shipment Quantity 装运数量		Actual Inspection Quantity 实际检验数量	Units Packed in Cartons 完成包装的件数		Units Finished Not Packed 未完成包装的件数		Units Not Finished 未完货的件数	
		Units 件数	Cartons 箱数		Qty 数量	%	Qty 数量	%	Qty 数量	%
XTDC-2022-FF02	1000000pcs (20000boxes)	1000000pcs (20000boxes)	500	315 boxes	1000000	100	0	0	0	0
Total 总数:	1000000pcs (20000boxes)	1000000pcs (20000boxes)	500	315 boxes	1000000	100	0	0	0	0

The above reflects our findings at the time and place of inspection based on comparison of your sample with random samples selected in accordance with a previously agreed random statistical standard. This report/certificate does not discharge or release sellers/suppliers from their commercial, legal or contractual obligations with regard to quality/quantity of delivery nor does it prejudice the buyer's right of claim toward sellers/suppliers for compensation for any apparent and/hidden defects not detected during our random inspection or occurring thereafter. 上述检查发现反映了在检验时间和地点的基于按照选定的随机抽样样品进行比较和预定的随机统计标准的结果。本报告/证书不能解除或减轻卖方和供应商的商业、法律或合同义务；不影响买方对交付产品的质量/数量过程中存在的以任何明显的和隐藏缺陷向卖方/供应商的索赔。无论是在我们随机检查或此后交付的过程中。

<b>INSPECTION REPORT</b> 检验报告 <i>THIS DOCUMENT IS NON-NEGOTIABLE</i> 此份报告不作为议单据	<b>Inspection No.</b> 检验编号: <b>3264758</b>	
	<b>Trade Assurance No.</b> 信保订单号: <b>197167851501022392</b>	
	<b>Report Date</b> 报告日期: <b>Sep 14, 2024</b>	<b>Page 3 of 17</b>

**G .PHOTOGRAPHS 照片**

	
1_Ware storage	2_Ware storage
	
3_Export carton	4_Export carton
	
5_Shipping mark	6_Shipping mark

The above reflects our findings at the time and place of inspection based on comparison of your sample with random samples selected in accordance with a previously agreed random statistical standard. This report/certificate does not discharge or release sellers/suppliers from their commercial, legal or contractual obligations with regard to quality/quantity of delivery nor does it prejudice the buyer's right of claim toward sellers/suppliers for compensation for any apparent and/hidden defects not detected during our random inspection or occurring thereafter. 上述检查发现反映了在检验时间和地点的基于按照选定的随机抽样样品进行比较和预定的随机统计标准的结果。本报告/证书不能解除或减轻卖方和供应商的商业、法律或合同义务；不影响买方对交付产品的质量/数量过程中存在的以任何明显的和隐藏缺陷向卖方/供应商的索赔。无论是在我们随机检查或此后交付的过程中。

<b>INSPECTION REPORT</b> 检验报告 <i>THIS DOCUMENT IS NON-NEGOTIABLE</i> 此份报告不作为议单据	<b>Inspection No.</b> 检验编号: <b>3264758</b>	
	<b>Trade Assurance No.</b> 信保订单号: <b>197167851501022392</b>	
	<b>Report Date</b> 报告日期: <b>Sep 14, 2024</b>	<b>Page 4 of 17</b>



7\_Shipping mark



8\_Shipping mark



9\_Shipping mark



10\_Shipping mark



11\_Open carton view



12\_Carton corrugated view

The above reflects our findings at the time and place of inspection based on comparison of your sample with random samples selected in accordance with a previously agreed random statistical standard. This report/certificate does not discharge or release sellers/suppliers from their commercial, legal or contractual obligations with regard to quality/quantity of delivery nor does it prejudice the buyer's right of claim toward sellers/suppliers for compensation for any apparent and/hidden defects not detected during our random inspection or occurring thereafter. 上述检查发现反映了在检验时间和地点的基于按照选定的随机抽样样品进行比较和预定的随机统计标准的结果。本报告/证书不能解除或减轻卖方和供应商的商业、法律或合同义务；不影响买方对交付产品的质量/数量过程中存在的以任何明显的和隐藏缺陷向卖方/供应商的索赔。无论是在我们随机检查或此后交付的过程中。

<b>INSPECTION REPORT</b> 检验报告 <i>THIS DOCUMENT IS NON-NEGOTIABLE</i> 此份报告不作为对单据	<b>Inspection No.</b> 检验编号: <b>3264758</b>	
	<b>Trade Assurance No.</b> 信保订单号: <b>197167851501022392</b>	
	<b>Report Date</b> 报告日期: <b>Sep 14, 2024</b>	<b>Page 5 of 17</b>




13\_Gift box view



14\_Gift box view



15\_Gift box view

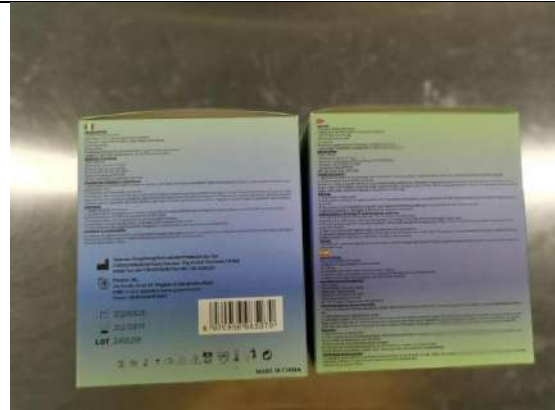

16\_Gift box view




17\_Gift box view




18\_Gift box view

The above reflects our findings at the time and place of inspection based on comparison of your sample with random samples selected in accordance with a previously agreed random statistical standard. This report/certificate does not discharge or release sellers/suppliers from their commercial, legal or contractual obligations with regard to quality/quantity of delivery nor does it prejudice the buyer's right of claim toward sellers/suppliers for compensation for any apparent and/hidden defects not detected during our random inspection or occurring thereafter. 上述检查发现反映了在检验时间和地点的基于按照选定的随机抽样样品进行比较和预定的随机统计标准的结果。本报告/证书不能解除或减轻卖方和供应商的商业、法律或合同义务；不影响买方对交付产品的质量/数量过程中存在的以任何明显的和隐藏缺陷向卖方/供应商的索赔。无论是在我们随机检查或此后交付的过程中。


<b>INSPECTION REPORT</b> 检验报告 <i>THIS DOCUMENT IS NON-NEGOTIABLE</i> 此份报告不作为议单据	<b>Inspection No.</b> 检验编号:	<b>3264758</b>	
	<b>Trade Assurance No.</b> 信保订单号:	<b>197167851501022392</b>	
	<b>Report Date</b> 报告日期:	<b>Sep 14, 2024</b>	<b>Page 6 of 17</b>

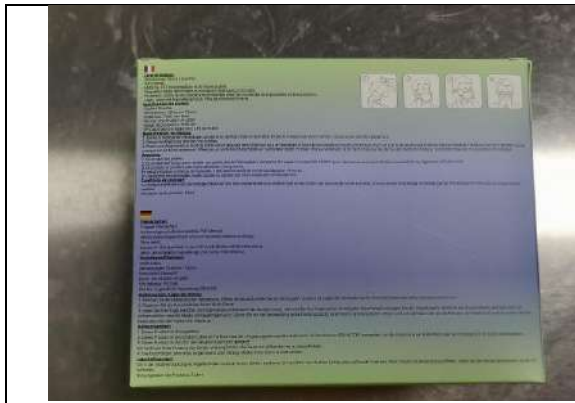


19\_Gift box view

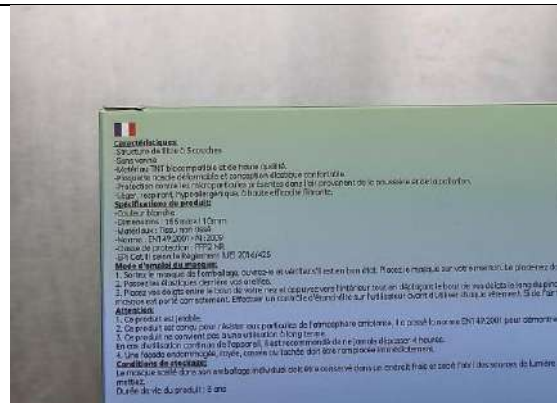


20\_Gift box view

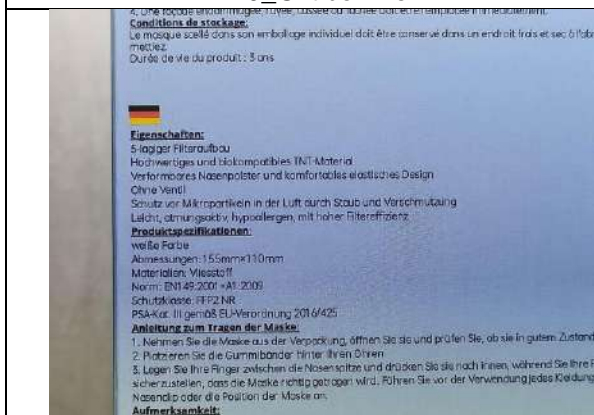


21\_Gift box view

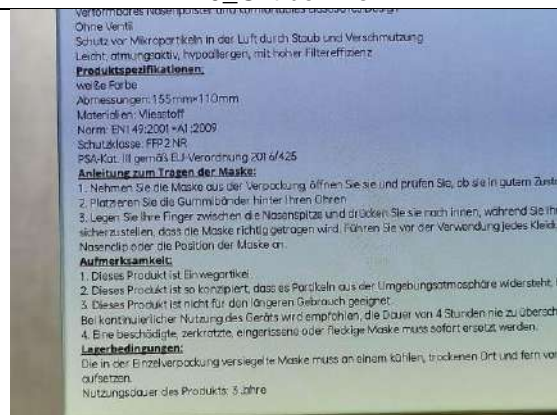

22\_Gift box view

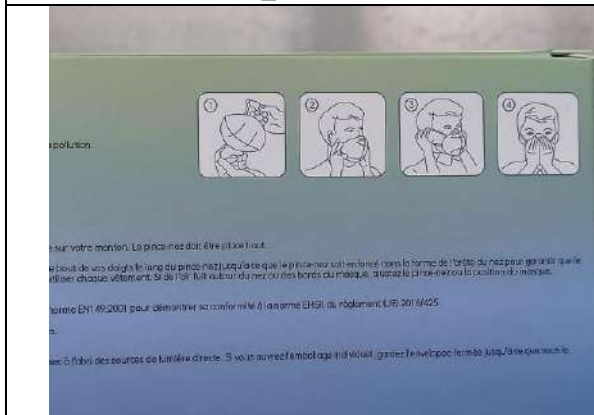


23\_Gift box view

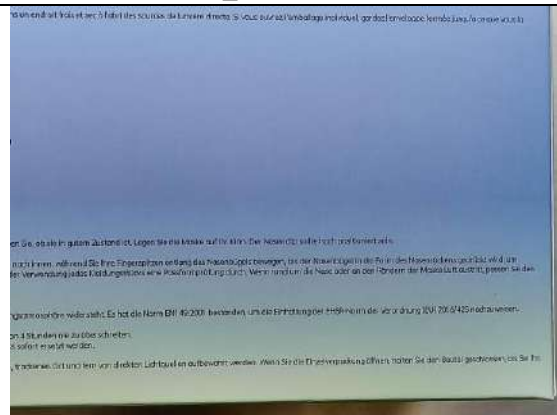


24\_Gift box view

The above reflects our findings at the time and place of inspection based on comparison of your sample with random samples selected in accordance with a previously agreed random statistical standard. This report/certificate does not discharge or release sellers/suppliers from their commercial, legal or contractual obligations with regard to quality/quantity of delivery nor does it prejudice the buyer's right of claim toward sellers/suppliers for compensation for any apparent and/hidden defects not detected during our random inspection or occurring thereafter. 上述检查发现反映了在检验时间和地点的基于按照选定的随机抽样样品进行比较和预定的随机统计标准的结果。本报告/证书不能解除或减轻卖方和供应商的商业、法律或合同义务；不影响买方对交付产品的质量/数量过程中存在的以任何明显的和隐藏缺陷向卖方/供应商的索赔。无论是在我们随机检查或此后交付的过程中。


<b>INSPECTION REPORT</b> 检验报告 <i>THIS DOCUMENT IS NON-NEGOTIABLE</i> 此报告不作为议单据	<b>Inspection No.</b> 检验编号:	<b>3264758</b>	
	<b>Trade Assurance No.</b> 信保订单号:	<b>197167851501022392</b>	
	<b>Report Date</b> 报告日期:	<b>Sep 14, 2024</b>	<b>Page 7 of 17</b>

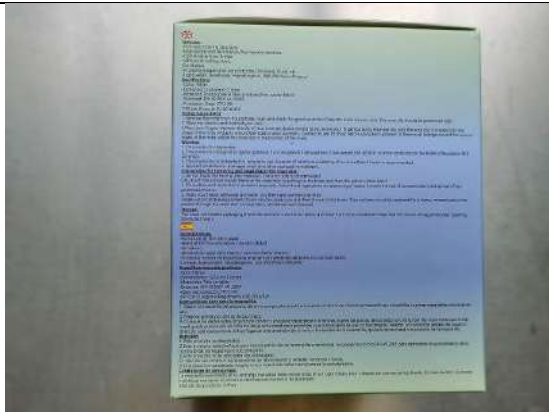


25\_Gift box view

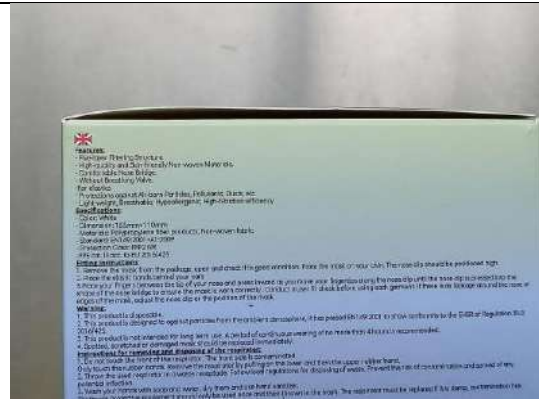

26\_Gift box view

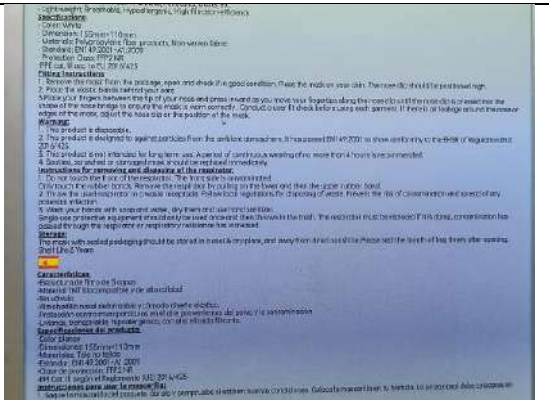

27\_Gift box view

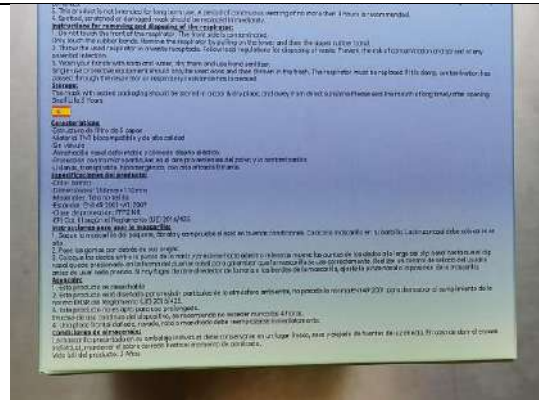

28\_Gift box view




29\_Gift box view




30\_Gift box view

The above reflects our findings at the time and place of inspection based on comparison of your sample with random samples selected in accordance with a previously agreed random statistical standard. This report/certificate does not discharge or release sellers/suppliers from their commercial, legal or contractual obligations with regard to quality/quantity of delivery nor does it prejudice the buyer's right of claim toward sellers/suppliers for compensation for any apparent and/hidden defects not detected during our random inspection or occurring thereafter. 上述检查发现反映了在检验时间和地点的基于按照选定的随机抽样样品进行比较和预定的随机统计标准的结果。本报告/证书不能解除或减轻卖方和供应商的商业、法律或合同义务；不影响买方对交付产品的质量/数量过程中存在的以任何明显的和隐藏缺陷向卖方/供应商的索赔。无论是在我们随机检查或此后交付的过程中。

<b>INSPECTION REPORT</b> 检验报告 <i>THIS DOCUMENT IS NON-NEGOTIABLE</i> 此份报告不作为议单据	<b>Inspection No.</b> 检验编号: <b>3264758</b>	
	<b>Trade Assurance No.</b> 信保订单号: <b>197167851501022392</b>	
	<b>Report Date</b> 报告日期: <b>Sep 14, 2024</b>	<b>Page 8 of 17</b>

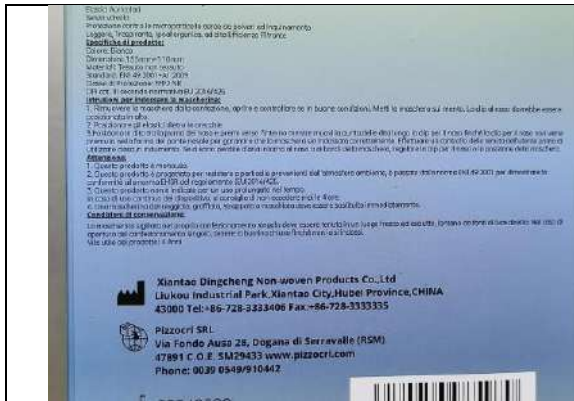

31\_Gift box view

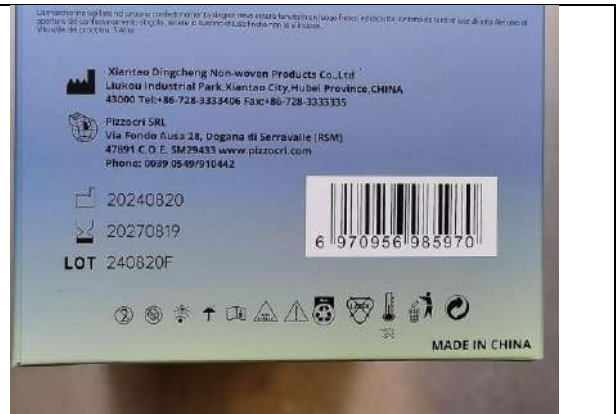

32\_Gift box view




33\_Barcode check



34\_Gift box view



35\_Gift box view

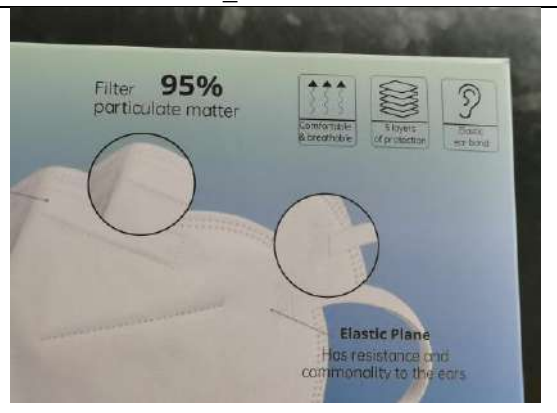

36\_Gift box view

The above reflects our findings at the time and place of inspection based on comparison of your sample with random samples selected in accordance with a previously agreed random statistical standard. This report/certificate does not discharge or release sellers/suppliers from their commercial, legal or contractual obligations with regard to quality/quantity of delivery nor does it prejudice the buyer's right of claim toward sellers/suppliers for compensation for any apparent and/hidden defects not detected during our random inspection or occurring thereafter. 上述检查发现反映了在检验时间和地点的基于按照选定的随机抽样样品进行比较和预定的随机统计标准的结果。本报告/证书不能解除或减轻卖方和供应商的商业、法律或合同义务；不影响买方对交付产品的质量/数量过程中存在的以任何明显的和隐藏缺陷向卖方/供应商的索赔。无论是在我们随机检查或此后交付的过程中。

<b>INSPECTION REPORT</b> 检验报告 <i>THIS DOCUMENT IS NON-NEGOTIABLE</i> 此份报告不作为对单据	<b>Inspection No.</b> 检验编号: <b>3264758</b>	
	<b>Trade Assurance No.</b> 信保订单号: <b>197167851501022392</b>	
	<b>Report Date</b> 报告日期: <b>Sep 14, 2024</b>	<b>Page 9 of 17</b>



37\_Open gift box view



38\_Open gift box view



39\_Manufacturer certificate



40\_Gift box packing view



41\_Unit packing view



42\_Unit packing view

The above reflects our findings at the time and place of inspection based on comparison of your sample with random samples selected in accordance with a previously agreed random statistical standard. This report/certificate does not discharge or release sellers/suppliers from their commercial, legal or contractual obligations with regard to quality/quantity of delivery nor does it prejudice the buyer's right of claim toward sellers/suppliers for compensation for any apparent and/hidden defects not detected during our random inspection or occurring thereafter. 上述检查发现反映了在检验时间和地点的基于按照选定的随机抽样样品进行比较和预定的随机统计标准的结果。本报告/证书不能解除或减轻卖方和供应商的商业、法律或合同义务；不影响买方对交付产品的质量/数量过程中存在的以任何明显的和隐藏缺陷向卖方/供应商的索赔。无论是在我们随机检查或此后交付的过程中。

<b>INSPECTION REPORT</b> 检验报告 <i>THIS DOCUMENT IS NON-NEGOTIABLE</i> 此份报告不作为对单据	<b>Inspection No.</b> 检验编号: <b>3264758</b>	
	<b>Trade Assurance No.</b> 信保订单号: <b>197167851501022392</b>	
	<b>Report Date</b> 报告日期: <b>Sep 14, 2024</b>	<b>Page 10 of 17</b>



43\_Product view




44\_Product view



45\_Product view



46\_Product view

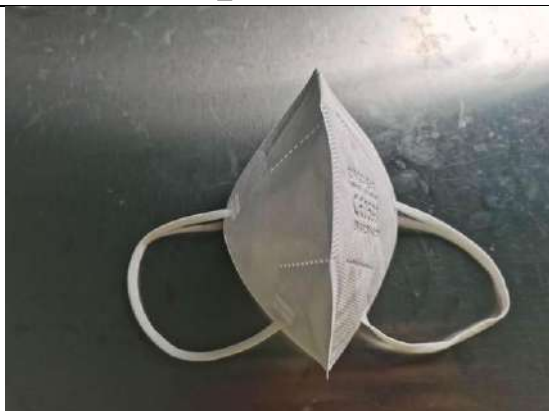

47\_Product view



48\_Product view

The above reflects our findings at the time and place of inspection based on comparison of your sample with random samples selected in accordance with a previously agreed random statistical standard. This report/certificate does not discharge or release sellers/suppliers from their commercial, legal or contractual obligations with regard to quality/quantity of delivery nor does it prejudice the buyer's right of claim toward sellers/suppliers for compensation for any apparent and/hidden defects not detected during our random inspection or occurring thereafter. 上述检查发现反映了在检验时间和地点的基于按照选定的随机抽样样品进行比较和预定的随机统计标准的结果。本报告/证书不能解除或减轻卖方和供应商的商业、法律或合同义务；不影响买方对交付产品的质量/数量过程中存在的以任何明显的和隐藏缺陷向卖方/供应商的索赔。无论是在我们随机检查或此后交付的过程中。

<b>INSPECTION REPORT</b> 检验报告 <i>THIS DOCUMENT IS NON-NEGOTIABLE</i> 此份报告不作为议单据	<b>Inspection No.</b> 检验编号: <b>3264758</b>	
	<b>Trade Assurance No.</b> 信保订单号: <b>197167851501022392</b>	
	<b>Report Date</b> 报告日期: <b>Sep 14, 2024</b>	<b>Page 11 of 17</b>



49\_Product view



50\_Size check



51\_Size check



52\_Size check



53\_Size check



54\_Size check

The above reflects our findings at the time and place of inspection based on comparison of your sample with random samples selected in accordance with a previously agreed random statistical standard. This report/certificate does not discharge or release sellers/suppliers from their commercial, legal or contractual obligations with regard to quality/quantity of delivery nor does it prejudice the buyer's right of claim toward sellers/suppliers for compensation for any apparent and/hidden defects not detected during our random inspection or occurring thereafter. 上述检查发现反映了在检验时间和地点的基于按照选定的随机抽样样品进行比较和预定的随机统计标准的结果。本报告/证书不能解除或减轻卖方和供应商的商业、法律或合同义务；不影响买方对交付产品的质量/数量过程中存在的以任何明显的和隐藏缺陷向卖方/供应商的索赔。无论是在我们随机检查或此后交付的过程中。

<b>INSPECTION REPORT</b> 检验报告 <i>THIS DOCUMENT IS NON-NEGOTIABLE</i> 此份报告不作为证据	<b>Inspection No.</b> 检验编号: <b>3264758</b>	
	<b>Trade Assurance No.</b> 信保订单号: <b>197167851501022392</b>	
	<b>Report Date</b> 报告日期: <b>Sep 14, 2024</b>	<b>Page 12 of 17</b>




55\_Size check



56\_Size check



57\_Size check

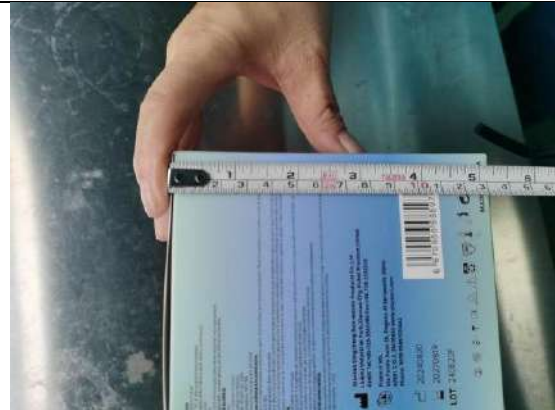

58\_Size check



59\_Size check



60\_Size check

The above reflects our findings at the time and place of inspection based on comparison of your sample with random samples selected in accordance with a previously agreed random statistical standard. This report/certificate does not discharge or release sellers/suppliers from their commercial, legal or contractual obligations with regard to quality/quantity of delivery nor does it prejudice the buyer's right of claim toward sellers/suppliers for compensation for any apparent and/hidden defects not detected during our random inspection or occurring thereafter. 上述检查发现反映了在检验时间和地点的基于按照选定的随机抽样样品进行比较和预定的随机统计标准的结果。本报告/证书不能解除或减轻卖方和供应商的商业、法律或合同义务；不影响买方对交付产品的质量/数量过程中存在的以任何明显的和隐藏缺陷向卖方/供应商的索赔。无论是在我们随机检查或此后交付的过程中。

<b>INSPECTION REPORT</b> 检验报告 <i>THIS DOCUMENT IS NON-NEGOTIABLE</i> 此份报告不作为对单据	<b>Inspection No.</b> 检验编号: <b>3264758</b>	
	<b>Trade Assurance No.</b> 信保订单号: <b>197167851501022392</b>	
	<b>Report Date</b> 报告日期: <b>Sep 14, 2024</b>	<b>Page 13 of 17</b>



61\_Size check



62\_Size check



63\_Size check



64\_Nose clip length check



65\_Nose clip length check




66\_Function check

The above reflects our findings at the time and place of inspection based on comparison of your sample with random samples selected in accordance with a previously agreed random statistical standard. This report/certificate does not discharge or release sellers/suppliers from their commercial, legal or contractual obligations with regard to quality/quantity of delivery nor does it prejudice the buyer's right of claim toward sellers/suppliers for compensation for any apparent and/hidden defects not detected during our random inspection or occurring thereafter. 上述检查发现反映了在检验时间和地点的基于按照选定的随机抽样样品进行比较和预定的随机统计标准的结果。本报告/证书不能解除或减轻卖方和供应商的商业、法律或合同义务；不影响买方对交付产品的质量/数量过程中存在的以任何明显的和隐藏缺陷向卖方/供应商的索赔。无论是在我们随机检查或此后交付的过程中。

<b>INSPECTION REPORT</b> 检验报告 <i>THIS DOCUMENT IS NON-NEGOTIABLE</i> 此份报告不作为议单据	<b>Inspection No.</b> 检验编号: <b>3264758</b>	
	<b>Trade Assurance No.</b> 信保订单号: <b>197167851501022392</b>	
	<b>Report Date</b> 报告日期: <b>Sep 14, 2024</b>	<b>Page 14 of 17</b>




67\_Function check

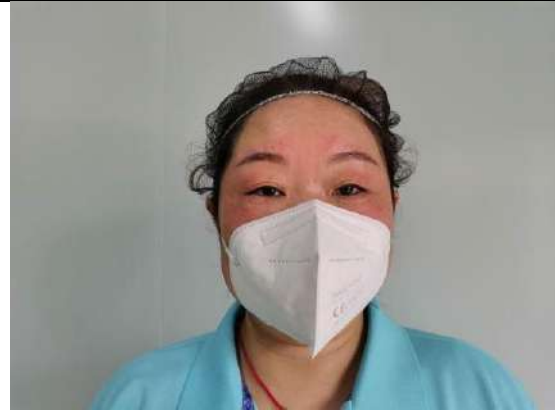

68\_Practical check



69\_Practical check

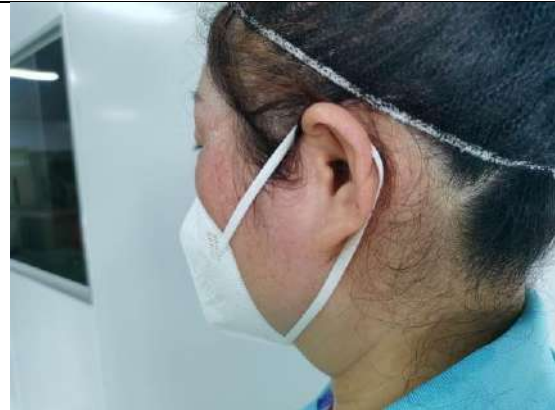

70\_Practical check



71\_Practical check




72\_Practical check

The above reflects our findings at the time and place of inspection based on comparison of your sample with random samples selected in accordance with a previously agreed random statistical standard. This report/certificate does not discharge or release sellers/suppliers from their commercial, legal or contractual obligations with regard to quality/quantity of delivery nor does it prejudice the buyer's right of claim toward sellers/suppliers for compensation for any apparent and/hidden defects not detected during our random inspection or occurring thereafter. 上述检查发现反映了在检验时间和地点的基于按照选定的随机抽样样品进行比较和预定的随机统计标准的结果。本报告/证书不能解除或减轻卖方和供应商的商业、法律或合同义务；不影响买方对交付产品的质量/数量过程中存在的以任何明显的和隐藏缺陷向卖方/供应商的索赔。无论是在我们随机检查或此后交付的过程中。

<b>INSPECTION REPORT</b> 检验报告 <i>THIS DOCUMENT IS NON-NEGOTIABLE</i> 此份报告不作为对单据	<b>Inspection No.</b> 检验编号: <b>3264758</b>	
	<b>Trade Assurance No.</b> 信保订单号: <b>197167851501022392</b>	
	<b>Report Date</b> 报告日期: <b>Sep 14, 2024</b>	<b>Page 15 of 17</b>



73\_Water penetration check check of filter

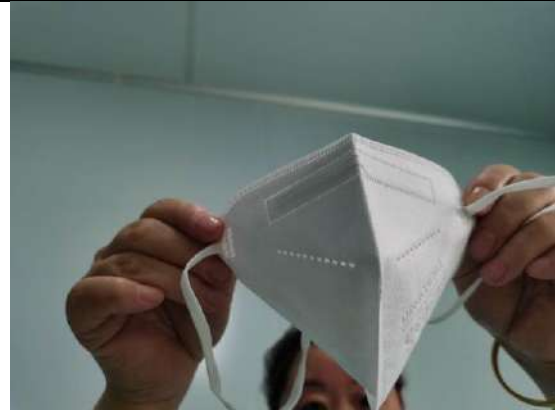

74\_Water penetration check check of filter



75\_No. of layer check



76\_Weight check



77\_Weight check



78\_Weight check

The above reflects our findings at the time and place of inspection based on comparison of your sample with random samples selected in accordance with a previously agreed random statistical standard. This report/certificate does not discharge or release sellers/suppliers from their commercial, legal or contractual obligations with regard to quality/quantity of delivery nor does it prejudice the buyer's right of claim toward sellers/suppliers for compensation for any apparent and/hidden defects not detected during our random inspection or occurring thereafter. 上述检查发现反映了在检验时间和地点的基于按照选定的随机抽样样品进行比较和预定的随机统计标准的结果。本报告/证书不能解除或减轻卖方和供应商的商业、法律或合同义务；不影响买方对交付产品的质量/数量过程中存在的以任何明显的和隐藏缺陷向卖方/供应商的索赔。无论是在我们随机检查或此后交付的过程中。

<b>INSPECTION REPORT</b> 检验报告 <i>THIS DOCUMENT IS NON-NEGOTIABLE</i> 此份报告不作为议单据	<b>Inspection No.</b> 检验编号: <b>3264758</b>	
	<b>Trade Assurance No.</b> 信保订单号: <b>197167851501022392</b>	
	<b>Report Date</b> 报告日期: <b>Sep 14, 2024</b>	<b>Page 16 of 17</b>



79\_Weight check



80\_Weight check



81\_Weight check



82\_Weight check



83\_Weight check



84\_Weight check

The above reflects our findings at the time and place of inspection based on comparison of your sample with random samples selected in accordance with a previously agreed random statistical standard. This report/certificate does not discharge or release sellers/suppliers from their commercial, legal or contractual obligations with regard to quality/quantity of delivery nor does it prejudice the buyer's right of claim toward sellers/suppliers for compensation for any apparent and/hidden defects not detected during our random inspection or occurring thereafter. 上述检查发现反映了在检验时间和地点的基于按照选定的随机抽样样品进行比较和预定的随机统计标准的结果。本报告/证书不能解除或减轻卖方和供应商的商业、法律或合同义务；不影响买方对交付产品的质量/数量过程中存在的以任何明显的和隐藏缺陷向卖方/供应商的索赔。无论是在我们随机检查或此后交付的过程中。

<b>INSPECTION REPORT</b> 检验报告 <i>THIS DOCUMENT IS NON-NEGOTIABLE</i> 此份报告不作为议单据	<b>Inspection No.</b> 检验编号: <b>3264758</b>	
	<b>Trade Assurance No.</b> 信保订单号: <b>197167851501022392</b>	
	<b>Report Date</b> 报告日期: <b>Sep 14, 2024</b>	<b>Page 17 of 17</b>



85\_defect-Insecure loop-ma



86\_defect-Dirt mark at loop-mi



87\_defect-Dirt mark at loop-mi



88\_defect-Dirt mark at loop-mi

The above reflects our findings at the time and place of inspection based on comparison of your sample with random samples selected in accordance with a previously agreed random statistical standard. This report/certificate does not discharge or release sellers/suppliers from their commercial, legal or contractual obligations with regard to quality/quantity of delivery nor does it prejudice the buyer's right of claim toward sellers/suppliers for compensation for any apparent and/hidden defects not detected during our random inspection or occurring thereafter. 上述检查发现反映了在检验时间和地点的基于按照选定的随机抽样样品进行比较和预定的随机统计标准的结果。本报告/证书不能解除或减轻卖方和供应商的商业、法律或合同义务；不影响买方对交付产品的质量/数量过程中存在的以任何明显的和隐藏缺陷向卖方/供应商的索赔。无论是在我们随机检查或此后交付的过程中。

仙桃市鼎成无纺布制品有限公司

Xiantao Dingcheng Non-woven Products Co.,Ltd

Liukou Industrial Park,Xiantao City,Hubei Province,CHINA 43000

[Tel:+86-18971459684](tel:+86-18971459684) [Fax:+86-728-3333335](tel:+86-728-3333335)

---

## DECLARATION OF CONFORMITY

We, **XIANTAO DINGCHENG NON-WOVEN PRODUCT CO.,LTD**, hereby declare that the below mentioned device complies with the applicable essential health and safety requirements of Regulation (EU) 2016/425 and standard(s) mentioned below .

**MODULE B 0598/PPE/22/3408** shall be used in conjunction with conformity assessment procedure module C2 or D.

***Name and Address of Product Owner:***

XIANTAO DINGCEHNG NON-WOVEN PRODUCT CO.,LTD  
Liukou Industrial Park, Xiantao City, Hubei Province, China 433000

***Name and Address of Physical Manufacturer:***

XIANTAO DINGCEHNG NON-WOVEN PRODUCT CO.,LTD  
Liukou Industrial Park, Xiantao City, Hubei Province, China 433000

***Name and Address of Authorized Representative:***

Pizzocri S.r.l  
Via Fondo Ausa , 28 Dogana di Serravalle Republic of San Marino Zip code : 47891

***Medical Device(s):*** Disposable face mask FFP2 NR

***Model :*** XTDC-2022-FF02 Filtering half mask

***Risk Classification:*** FFP2 NR

***Quality Management System Certificate:*** ISO13485


***Standards Applied:*** EN 149:2001+A1:2009

***This declaration of conformity is valid from 2022.09.02***

***Authorized Signator***

\_\_\_\_\_  
Name Position

\_\_\_\_\_  
Date

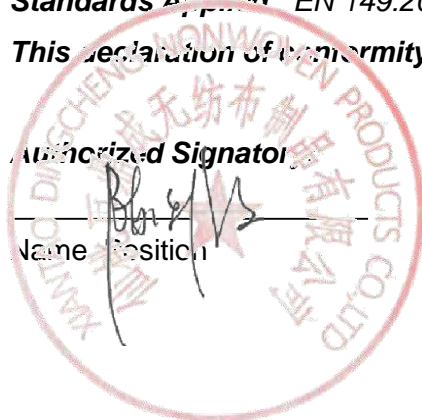

# Mascherine FFP2 NR

Senza valvola - Elastici Auricolari  
Certificate come DPI di III Categoria



Filter **95%**  
particulate matter

REF: XTDC-2022-FF02

LOT 240820F

20/08/2024

20/08/2027

**CE 0598**

2000 pcs

QTY: 40pcs/box 50 boxes/cnt



Xiantao Dingcheng Nonwoven Products Co., Ltd.  
Liukou Industrial Park, Xiantao, Hubei, China.



Pizzocri S.r.l.  
Via Fondo Ausa, 28 Dogana (RSM) San Marino 47891  
C.F. 91187940407 C.O.E. 29433 - n. iscr. 8747  
Tel. 0549910442 - pizzocri@pec.sm - www.pizzocri.com

EN 149:2001A1:2009  
PPE REGULATION EU 2016/425

Made in China

# Mascherine FFP2 NR

Senza valvola - Elastici Auricolari  
Certificate come DPI di III Categoria



Filter **95%**  
particulate matter

REF: XTDC-2022-FF02



2000 pcs

QTY: 40pcs/box 50 boxes/cnt

Size:155mm×110mm  
G.W.: 15 KGS  
N.W.: 14 KGS  
MEAS:57\*33\*67cm

EN 149:2001A1:2009  
PPE REGULATION

Made in China





**DINGCHENG**

FFP2 NR XTDC-2022-FF02

**CE 0598**

EN 149:2001+A1:2009



DINGCHENG

FFP1 NR X7DC-030A/FFP1

CE 0598

EN 149:2001+A1:2009

**FFP2 NR**

Model: XTDC-2022-FF02



EN 149:2001+A1:2009  
PPE REGULATION EU 2016/425

CND : T0399  
RDM : 2514990

**CE 0598**

40  
PCS

Filter **95%**  
particulate matter



**Nasal Regulator**  
Perfect fitting  
nose pair



**Effective Filtering**  
Filter 95%  
of particles

**Elastic Plane**  
Has resistance and  
commonality to the ears

**Soft Touch**  
Apply soft burning  
tissue to improve  
skin contact.

EN 149:2001 +A1:2009  
PPE REGULATION EU 2016/425

**CE 0598**

NON-MEDICAL USAGE  
FOR SINGLE USE ONLY  
EFFECTIVE AGAINST SOLID AND LIQUID AEROSOLS







**Features:**

- Five-layer Filtering Structure.
- High-quality and Skin-friendly Non-woven Materials.
- Comfortable Nose Bridge.
- Without Breathing Valve.
- Ear elastics
- Protections against Air-born Particles, Pollutants, Dusts, etc.
- Light-weight, Breathable, Hypoallergenic, High-filtration-efficiency

**Specifications:**

- Color: White
- Dimension: 159.99mm x 104.44mm
- Materials: Polypropylene fiber products, Non-woven fabric
- Standard: EN149:2001+A1:2009
- Protection Class: FFP2 NR
- PPE cat. III acc. to EU 2016/425

**Fitting Instructions:**

1. Remove the mask from the package, open and check if in good condition. Place the mask on your chin. The nose clip should be positioned high.
2. Place the elastic bands behind your ears
3. Place your fingers between the tip of your nose and press inward as you move your fingertips along the nose clip until the nose clip is pressed into the shape of the nose bridge to ensure the mask is worn correctly. Conduct a user fit check before using each garment. If there is air leakage around the nose or edges of the mask, adjust the nose clip or the position of the mask.

**Warnings:**

1. This product is disposable.
2. This product is designed to against particles from the ambient atmosphere, It has passed EN149:2001 to show conformity to the EHSR of Regulation (EU) 2016/425.
3. This product is not intended for long term use. A period of continuous wearing of no more than 4 hours is recommended.
4. Spotted, scratched or damaged mask should be replaced immediately.

**Instructions for removing and disposing of the respirator:**

1. Do not touch the front of the respirator. The front side is contaminated. Only touch the rubber bands. Remove the respirator by pulling on the lower and then the upper rubber band.
  2. Throw the used respirator in a waste receptacle. Follow local regulations for disposing of waste. Prevent the risk of contamination and spread of any potential infection.
  3. Wash your hands with soap and water, dry them and use hand sanitizer.
- Single-use protective equipment should only be used once and then thrown in the trash. The respirator must be replaced if it is damp, contamination has passed through the respirator or respiratory resistance has increased.

**Storage:**

The mask with sealed packaging should be stored in a cool & dry place, and away from direct sunshine. Please seal the mouth of bag timely after opening.

Shelf Life: 3 Years

**Características:**

- Estructura de filtro de 5 capas
- Material TNT biocompatible y de alta calidad
- Sin válvula
- Almohadilla nasal deformable y cómodo diseño elástico.
- Protección contra micropartículas en el aire provenientes del polvo y la contaminación.
- Livianas, transpirable, hipotérgico, con alta eficacia filtrante.

**Especificaciones del producto:**

- Color: blanco
- Dimensiones: 159,99 mm x 104,44 mm
- Materiales: Tela no tejida
- Estándar: EN149:2001+A1:2009
- Clase de protección: FFP2 NR
- EPI Cat. III según el Reglamento (UE) 2016/425

**Instrucciones para usar la mascarilla:**

1. Saque la mascarilla del paquete, ábrala y compruebe si está en buenas condiciones. Coloque la mascarilla en tu barbilla. La pinza nasal debe colocarse en alto.
2. Pase las gomas por detrás de sus orejas.
3. Coloque los dedos entre la punta de la nariz y presione hacia adentro mientras mueve las puntas de los dedos a lo largo del clip nasal hasta que el clip nasal quede presionado en la forma del puente nasal para garantizar que la mascarilla se use correctamente. Realizar un control de sellado del usuario antes de usar cada prenda. Si hay fugas de aire alrededor de la nariz o los bordes de la mascarilla, ajuste la pinza nasal o la posición de la mascarilla.

**Atención:**

1. Este producto es desechable.
  2. Este producto está diseñado para resistir partículas de la atmósfera ambiental, ha pasado la norma EN149:2001 para demostrar el cumplimiento de la norma EHSR del Reglamento (UE) 2016/425.
  3. Este producto no es apto para uso prolongado.
  4. Una placa frontal dañada, rayada, rota o manchada debe reemplazarse inmediatamente.
- En caso de uso continuo del dispositivo, se recomienda no exceder nunca las 4 horas.

**Condiciones de almacenaje:**

La mascarilla preimpresa en su embalaje individual debe conservarse en un lugar fresco, seco y alejado de fuentes de luz directa. En caso de abrir el envase individual, mantener el sobre cerrado hasta el momento de ponérselo.

Vida útil del producto: 3 Años

**Caractéristiques:**

- Structure de filtre à 5 couches
- Sans valve
- Matériau TNT biocompatible et de haute qualité.
- Plaque nasale déformable et conception élastique confortable.
- Protection contre les microparticules présentes dans l'air provenant de la poussière et de la pollution.
- Léger, respirant, hypoallergénique, à haute efficacité filtrante.

**Spécifications du produit:**

- Couleur blanche
- Dimensions: 59,99 mm x 104,44 mm
- Matériaux: Tisseu non tissé
- Norme: EN149:2001 +A1:2009
- Classe de protection: FFP2 NR
- EPI Cat. III selon le Règlement (UE) 2016/425

**Mode d'emploi du masque:**

- Sortez le masque de l'emballage, ouvrez-le et vérifiez s'il est en bon état. Placez le masque sur votre menton. Le pince-nez doit être placé haut.
- Passer les élastiques derrière vos oreilles.
- Placez vos doigts entre le bout de votre nez et appuyez vers l'intérieur tout en déplaçant le bout de vos doigts le long du pince-nez jusqu'à ce que le pince-nez soit enfoncé dans la forme de l'arête du nez pour garantir que le masque est porté correctement. Effectuer un contrôle d'étanchéité sur l'utilisateur avant d'utiliser chaque vêtement. Si de l'air fuit autour du nez ou des bords du masque, ajustez le pince-nez ou la position du masque.

**Attention:**

- Ce produit est jetable.
  - Ce produit est conçu pour résister aux particules de l'atmosphère ambiante, il a passé la norme EN149:2001 pour démontrer sa conformité à la norme EHSR du règlement (UE) 2016/425.
  - Ce produit ne convient pas à une utilisation à long terme.
- En cas d'utilisation continue de l'appareil, il est recommandé de ne jamais dépasser 4 heures.

- Une façade endommagée, rayée, cassée ou tachée doit être remplacée immédiatement.

**Conditions de stockage:**

Le masque scellé dans son emballage individuel doit être conservé dans un endroit frais et sec à l'abri des sources de lumière directe. Si vous ouvrez l'emballage individuel, gardez l'enveloppe fermée jusqu'à ce que vous la mettiez.

Durée de vie du produit: 3 ans

**Eigenschaften:**

- 5-lagiger Filteraufbau
- Hochwertiges und biokompatibles TNT-Material
- Verformbares Nasenpolster und komfortables elastisches Design
- Ohne Ventil
- Schutz vor Mikropartikeln in der Luft durch Staub und Verschmutzung
- Leicht, atmungsaktiv, hypoallergen, mit hoher Filtereffizienz

**Produktspezifikationen:**

- weisse Farbe
- Abmessungen: 159,99 mm x 104,44 mm
- Materialien: Vliesstoff
- Norm: EN149:2001 +A1:2009
- Schutzklasse: FFP2 NR
- PSA-Kat. III gemäß EU-Verordnung 2016/425

**Anleitung zum Tragen der Maske:**

- Nehmen Sie die Maske aus der Verpackung, öffnen Sie sie und prüfen Sie, ob sie in gutem Zustand ist. Legen Sie die Maske auf Ihr Kinn. Der Nasendip sollte hoch positioniert sein.
- Platzieren Sie die Gummibänder hinter Ihren Ohren
- Legen Sie Ihre Finger zwischen die Nasenspitze und drücken Sie sie nach innen, während Sie Ihre Fingerspitzen entlang des Nasenbügels bewegen, bis der Nasenbügel in die Form des Nasenrückens gedrückt wird, um sicherzustellen, dass die Maske richtig getragen wird. Führen Sie vor der Verwendung jedes Kleidungsstücks eine Passformprüfung durch. Wenn rund um die Nase oder an den Rändern der Maske Luft austritt, passen Sie den Nasendip oder die Position der Maske an.

**Aufmerksamkeiten:**

- Dieses Produkt ist Einwegartikel.
- Dieses Produkt ist so konzipiert, dass es Partikeln aus der Umgebungsluft widersteht. Es hat die Norm EN149:2001 bestanden, um die Einhaltung der EHSR-Norm der Verordnung (EU) 2016/425 nachzuweisen.
- Dieses Produkt ist nicht für den längeren Gebrauch geeignet.
- Bei kontinuierlicher Nutzung des Geräts wird empfohlen, die Dauer von 4 Stunden nie zu überschreiten.
- Eine beschädigte, zerkratzte, eingerissene oder fleckige Maske muss sofort ersetzt werden.

**Lagerbedingungen:**

Die in der Einzelverpackung versiegelte Maske muss an einem kühlen, trockenen Ort und fern von direkten Lichtquellen aufbewahrt werden. Wenn Sie die Einzelverpackung öffnen, halten Sie den Beutel geschlossen, bis Sie ihn aufsetzen.

Nutzungsdauer des Produkts: 3 Jahre